TESLA SMART BLOOD PRESSURE MONITOR

USER MANUAL

Model: U82RH

EN

DE

CZ

SK

PL

HU

SL

Please read the manual carefully before you use the unit and keep the manual well after using.

SAFETY INFORMATION

- Your new digital blood pressure monitor uses the oscillometric method of blood pressure measurement. This means the monitor detects your blood's movement through your brachial artery and converts the movements into a digital Reading. An oscillometric monitor does not need a stethoscope, so the monitor is simple to use.
- Intelligent inflation will reduce the uncomfortable feeling by incorrect inflation, and shorten the measurement time, prolong the cuff's usage lifetime.
- 2x199 sets memory function each measurement result will be displayed on the screen and automatically stored. This unit has blood classification index, could easy to check your blood pressure.
- This automatic blood pressure monitor could measure the systolic pressure diastolic pressure and pulse, the components are included the body, cuff and printed instruction manual. This unit is intended for adult use.

INTENDED USE

The automatic blood pressure monitor intended to measure the systolic pressure, diastolic pressure, and pulse rate through upper arm. They are expect used in the home, and hospital, intended for over 12 years old adult using.

Contraindication

This product can't be used in patients who is with severe heart insufficiency to avoid suffocation and death. This product is not suitable for infants and children.

- To assure the correct use at the product, basic safety measures should always be followed including the warning and the caution listed in the instruction manual:

 **WARNING:** This alert identifies hazards that may cause serious personal injury or death.

 **CAUTION:** This alert identifies hazards that may cause minor personal injury, product damage, or property damage.

SAFETY INFORMATION

 Those who have arrhythmia, diabetes, blood circulation or apoplexy problem please use under the physician's instruction.

 Contact your physician for specific information about your blood pressure. Self-diagnosis and Treatment which use measured results may be dangerous. Follow The instructions of your physician or licensed healthcare provider.

 Please place on a high place where it can't be touched by children.

 No modification of this equipment is allowed.

 Do not modify This equipment without authorization of the manufacturer.

 If this equipment is modified, appropriate inspection and testing must be conducted to ensure continued safe use of equipment.

 The cuff hose around the neck may cause the suffocation.

 The swallowing of small part like packing bag, battery, battery cover and so on may cause the suffocation.

 Please don't use a dilution agent, alcohol or, petrol to clean the unit. Please don't hit heavily or fall down the product from a high place. Use The right cuff, otherwise, it cannot work.

 Never leave any low battery in the battery compartment since they may leak and cause damage To The unit.

 Please take off the battery if you won't use device in 3 months.

 Replace The new batteries if the unit displays a low battery symbol.

 Do not mix the old and new batteries.

 Please avoid using in a high radiant area to make your measuring data correctly.

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