CANYON
YOU CAN

Fitness band
Smart coach SB-75

QUICK GUIDE v2

SMART COACH WITH ECG-BASED PERSONAL VIRTUAL COACH FUNCTION.

- ECG diagnostics
- Multi-sport mode
- 0,96 inch TFT screen
- IP67 waterproof rating
- Camera remote
- Compatibility with iOS and Android OS

Complete set

- Fitness band
- User manual

DEVICE OVERVIEW

1. Strap (removable strap)
2. ECG measurement contacts
3. TFT monitor
4. Sensor button (switch function, press and hold to confirm or exit the page)
5. Battery charging indicator
6. PPG sensor

INTRODUCTION

Thank you for purchasing the SB75 Smart Coach!

The SB75 is equipped with all the modern functions of a fitness bracelet (number of steps taken (pedometer), calories monitor, sleep duration, heart rate measurement, displaying new messages from the smartphone, etc.), but, in addition, it has a unique feature – measuring the ECG (electrocardiogram)!

Of course, this is not a medical device: the SB75 only measures the ECG in lead I - left / right hand. However, using innovative equipment and a unique approach, we learned how to get a lot of useful and necessary information for the SB75 user.

With the help of a high-frequency sensor (500 Hz), we obtain detailed high-resolution ECG images. Thanks to the increased memory and a cloud-based data solution, within seconds of completing a measurement, the SB75 user receives the processed results in the Canyon Life app in the form of clear graphs and values. The data obtained is the result of the use of unique mathematical algorithms for processing high-resolution ECG images. This is important because the algorithm can obtain more information from these pictures, not only the pulse.

At the moment (beginning of 2021), we have learned to decipher such important indicators of a person's health as his psycho-emotional and physiological state (readiness for cardio loads, stress level, etc.). In addition, the app will offer you expert advice on your workout or recovery. Also in the Canyon Life application you can save and share the ECG result as a graphic file.

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