# GIMA

PROFESSIONAL MEDICAL PRODUCTS

**MISURATORE DI PRESSIONE LEO CON SOFTWARE**  
**LEO BLOOD PRESSURE MONITOR**  
**WITH SOFTWARE**  
**TENSIOMÈTRE LEO AVEC LOGICIEL**  
**DE LEO MEDIDOR DE PRESIÓN**  
**CON SOFTWARE**  
**MEDIDOR DA TENSÃO LEO COM SOFTWARE**

REF

**32902 / CONTEC08A**

CONTEC MEDICAL SYSTEMS CO., LTD  
No.112 Qinhuang West Street, Economic & Technical Development Zone,  
Qinhuangdao, Hebei Province, PEOPLE'S REPUBLIC OF CHINA  
Made in China

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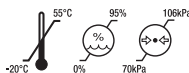
CE 0123

## Foreword

**Please read the User Manual carefully before using this product. The User Manual which describes the operating procedures should be followed strictly.** This manual detailed introduce the steps must be noted when using the product, operation which may result in abnormal, the risk may cause personal injury and product damage and other contents, refer to the chapters for details. Any anomalies or personal injury and device damage arising from use, maintain, store do not follow requirements of the User Manual.

**Note: Please read the User Manual carefully before using this product.**

Described in this User Manual is in accordance with practical situation of the product. In case of modifications and software upgrades, the information contained in this document is subject to change without notice.

## The warning items

**Before using this product, you should consider the safety and efficacy of the following described:**

- Described each measurement results combined with clinical symptoms by qualified doctors.
- The reliability and operation of using this product whether meets the operation of this manual relate to the maintenance instructions.
- The intended operator of this product may be the patient.
- Do not perform maintenance and service while the device is in use.

**Warning: Replace accessories which not provided by our company may lead to the occurrence of errors. Replace adapters, cuffs or SpO2 probes at will may result in wrong measurement results. Without our company or other approved maintenance organizations trained service personnel should not try to maintain the product.**

## **Contents**

<b>Chapter 1</b>	<b>Safety Precautions</b> .....	<b>53</b>
	1.1 Operation for AC Adapter (Separate Sale) .....	56
	1.2 Operation for Battery .....	56
<b>Chapter 2</b>	<b>Main Unit</b> .....	<b>58</b>
<b>Chapter 3</b>	<b>External Interfaces</b> .....	<b>61</b>
<b>Chapter 4</b>	<b>Battery/AC Adapter Installation</b> .....	<b>63</b>
	4.1 Battery Installation .....	63
	4.2 Usage of power adapter .....	64
<b>Chapter 5</b>	<b>Button Functions</b> .....	<b>64</b>
<b>Chapter 6</b>	<b>Setting the Date and Time</b> .....	<b>65</b>
<b>Chapter 7</b>	<b>About Unit</b> .....	<b>66</b>
<b>Chapter 8</b>	<b>User Switch</b> .....	<b>66</b>
<b>Chapter 9</b>	<b>Over-limit Prompt Function</b> .....	<b>67</b>
	9.1 Physiological parameter over-limit prompt .....	67
	9.2 Technical parameter over-limit prompt .....	68
<b>Chapter 10</b>	<b>The Usage Method of Sphygmomanometer</b> .....	<b>68</b>
	10.1 Accurate Measurement Way .....	68
	10.2 Applying the Cuff .....	70
	10.3 BP Measurement.....	72
<b>Chapter 11</b>	<b>Memory Function</b> .....	<b>73</b>
	11.1 Review Memory Values .....	73
	11.2 Delete Memory Values.....	73
<b>Chapter 12</b>	<b>SpO<sub>2</sub> Measurement Function</b> .....	<b>74</b>
<b>Chapter 13</b>	<b>SpO<sub>2</sub> Measurement Method</b> .....	<b>77</b>

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Shanghai International Holding Corp. GmbH (Europe)
**Gima S.p.A.**